# DUALSONIC

*Professional* | *Luxury*

CE

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***The best of HIFU technology***  
*for a youthful, smooth and suppled skin, a lifted and reshaped face or body.*

*Modular, precise, efficient and safe. No surgery, No unavailability !*

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### World's First Patent Dual-Layer HIFU Technology

DUALSONIC is a HIFU portable aesthetic device specially designed for professional or home treatment of the face and body. Effective, practical, precise, safe, the modular device allows to improve skin quality, reduce wrinkles, lift and remodel by acting on the different skin layers including SMAS. With high intensity focused ultrasound you get a more perfect and elastic skin.

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
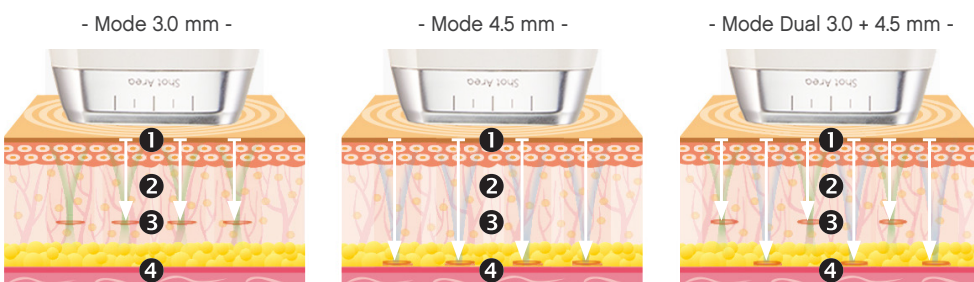
### → 2 HIFU CARTRIDGES for a global and safe Face treatment.

With an adjustable emission depth to 3.0 mm, 4.5 mm or DUAL (3.0 and 4.5 mm simultaneously), the HIFU procedure can be easily performed on the entire face without having to change cartridge. For more delicate periorbital areas, an exclusive Eye Cartridge using a 1.5 mm depth is proposed.

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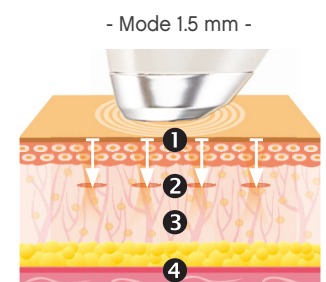→ A controlled HIFU energy emission dedicated to each face or eyes treated area.

Two dedicated cartridges types to treat each area. With a 3.0 mm, 4.5 mm or double (3.0 mm + 4.5 mm) emission depth face cartridge and 1.5 mm eye cartridge, it is possible to fight the skin aging effects, lift in depth, improve elasticity for a recovered skin quality.

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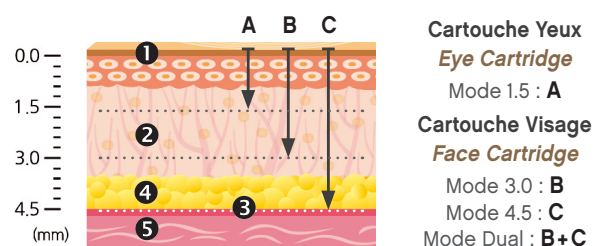
→ Essential role of SMAS in skin treatment.

SMAS is a layer of tissue 4.5mm deep under the skin between hypodermis and muscles. It is one of the most important support structures for the face. As the skin ages, the SMAS sags causing low cheeks, double chin, pockets, nasolabial folds... By repositioning the SMAS, the HIFU treatment corrects the effects and unsightliness related to its downward movement. Most naturally in the world, the burdened features are erased, without pulling effect on the skin since it only follows the ascent of the SMAS with which it is in close contact. Unlike laser or RF devices that do not reach the SMAS, with DUALSONIC you will be able to modify in depth the consequences of aging.

▶ Cartouches et couches traitées. *Cartridges and treated layers.*

1 - Épiderme (Epidermis). 2 - Derme (Dermis). 3 - SMAS (SMAS). 4 - Couche sous-cutanée (Subcutaneous layer), 5 - Muscle (Muscle).

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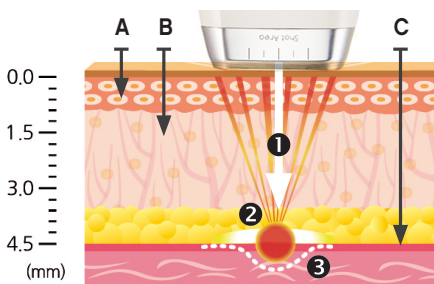
## → Principles and effects

Ultrasound is a harmless wave that has long been used in therapeutic medicine. DUALSONIC makes it possible to use ultrasonic energy for esthetic purposes by focusing and focusing these waves on specific points of the face in order to cause a rise in temperature.

The thermal stress applied at a predefined depth generates coagulation points on the different skin layers causing a natural production of collagen and elastin, shrinkage and regeneration of the tissues. The treatment allows to obtain a smoothing effect, a tension of the skin and improvement of its texture. During the process, energy passes through the epidermis to the dermis and SMAS without cutting or damaging sagging and wrinkled skin. and SMAS without cutting or damaging the sagged and wrinkled skin.

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Skin Layers treated by different therapeutic technologies :

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1 - Ultrasons focalisés (focused ultrasounds). 2 - Point de coagulation sur la couche du SMAS (Coagulation point on SMAS layer). 3 - Zone traitée (Treated zone).

Plis nasolabiaux, rides, poches sous les yeux, double menton, joues, les ultrasons appliqués permettent un véritable lifting et remodelage du visage. (Nasolabial folds, wrinkles, bags under the eyes, double chin, cheeks, the applied ultrasounds allow a real lifting and remodeling of the face).

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## → Contraindications :

Pregnant or breastfeeding women, systemic or local skin diseases (severe or cystic acne, severe chelloids, hernia, lesions or haemorrhages, swelling, infections or inflammations...), patients carrying pacemakers, implants or metallic devices.

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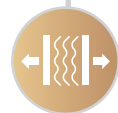
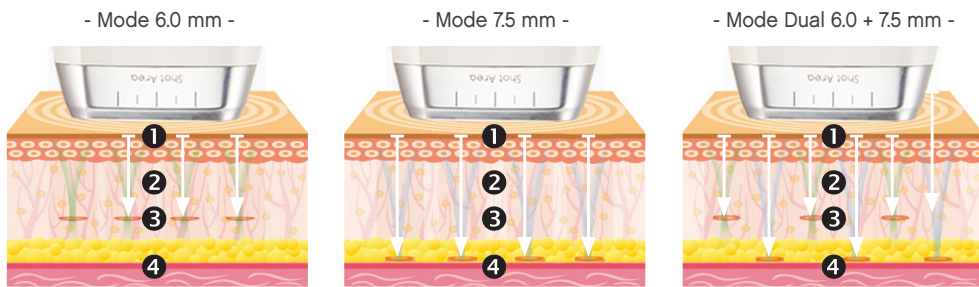
## → 1 HIFU CARTRIDGE for a specific and safe Body treatment. (option)

With an adjustable emission depth to 6.0 mm, 7.5 mm or DUAL (6.0 and 7.5 mm simultaneously), the HIFU procedure can be easily performed on the body without having to change cartridge.

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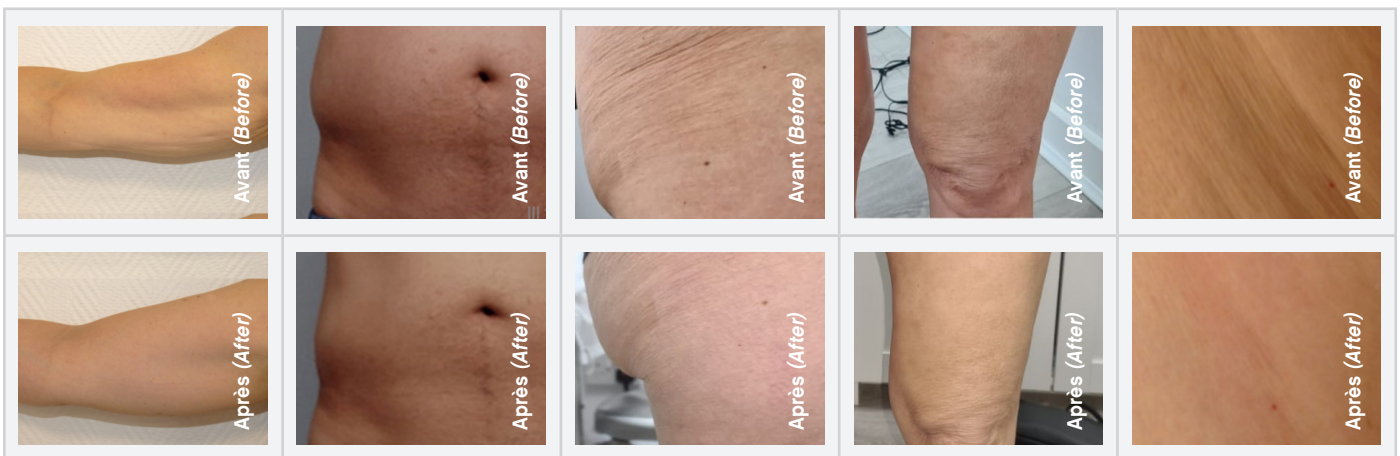
## → A controlled HIFU energy emission dedicated to each specific treated body area.

Easily scalable and configurable, the body cartridge allows you to approach each treatment and targeted area with the most relevant setting and meet the demand for efficiency and safety required by the patient.

With an emission capacity ranging from 6.0 to 7.5 mm, you can fight : Sagging, laxity of the inner surface of the arms and thighs, Loss of elasticity of the skin above the knees, Cellulite and body irregularities, Dégradations of the contour lines and volume in the waist and sides, Gravitational ptosis of abdominal tissues, Local fatty deposits in the abdomen, sides, Moderate sagging and excess skin after pregnancy, Laxity and wrinkling in the supra- and axillary area. It is possible to act on the thighs, buttocks, knees, back, arms, shoulders...

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